



**In This Issue:**

Special Edition:  
New Web/VS  
icons

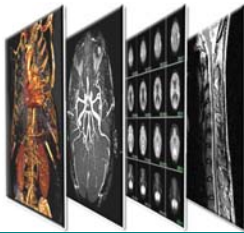

# EYE ON IMAGING

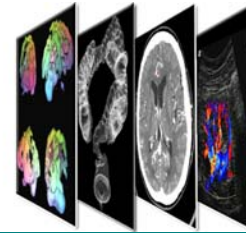

**C**ARE  
**S**ERVICE  
**I**NNOVATION

March, 2007



Cedars-Sinai Medical Center  
S. Mark Taper Foundation Imaging Center


Volume II  
Issue 1

## CURRENT ISSUES

## NEW WEB/VS ICONS FOR IMAGES

**Barry D. Pressman, MD, FACR**  
Chairman, S. Mark Taper Foundation  
Imaging Center

Click on tab names for more detailed information.

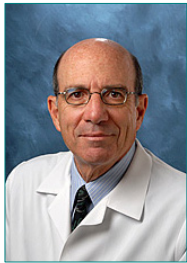

### WEB/VS ICONS

As many of you have already noticed there has been a recent upgrade to the PACS system. On Web/VS this allows for faster downloading of















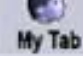

images, as well as for accepting larger image files.

Please note that the new system utilizes new icons which are illustrated and explained in this newsletter.

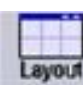






If you have any questions regarding utilizing this new system please contact:

- Mischa Harris: 423-4440 or
- Tim Fletcher and Barry Haggberg: 423-5500.

I welcome your input and suggestions. Please send them to me at [pressmanb@cshs.org](mailto:pressmanb@cshs.org).

Old Icon	Current Description	New Icon
	<b>File Tab</b> Save, E-mail, or print images.	
	<b>Edit Tab</b> Manipulate the selected images displayed in the viewer.	
	<b>Windowing Tab</b> Adjust the windowing values of the selected images, either automatically or interactively, as well as invert the gray levels of the images.	
	<b>Layout Tab</b> Configure the format in which the groups and images are displayed on the screen.	
	<b>Measurement Tab</b> Add and manipulate graphic and text annotations on images.	
	<b>Tools Tab</b> Show the reference lines in images, link images, and view the images in Cine mode.	
	<b>Conference Tab</b> This tab is not used.	
	<b>My Tab</b> Lets you contain your most useful tools. Right-click on any of the viewer tabs, select a tool, then click Add to My-tab. The selected tool appears in the My Tab Folder.	

### THE FOLLOWING ICONS ARE POSITIONED AT THE TOP OF THE TOOLBAR:

	<b>Hide Toolbar</b> This button makes the toolbar disappear. To bring the toolbar back, right click on the screen and select "Show/Hide Toolbar". (This icon is shown larger than actual size on the screen.)		<b>Help</b> This button pulls up the complete user's guide for Web/VS. (This icon is shown larger than actual size on the screen.)
	<b>Exit</b> This button returns you to the Web/VS screen. (This icon is shown larger than actual size on the screen.)		<b>Reports</b> This button pulls up the report which corresponds to the current exam.
	<b>Patient Mini Archive</b> This button pulls up a list of all available PACS images for the patient.		<b>Save</b> This button allows you to save images to your computer or disk.
	<b>Done</b> This button goes back to the list of the patient's available PACS images. Note: this button is not always available.	You can receive further information on our website: <a href="http://www.csmc.edu/imaging-md">www.csmc.edu/imaging-md</a>	